








# Adaptive CALL BELLS

Names	Picture Descriptions	Descriptions	Ideas for Use
<b>Breath Activated Call Activated nurse call</b>		<p>BreathCall is activated by simply breathing into a disposable straw and filter assembly. It has a flexible gooseneck which bends to almost any position to suit the user. universal mounting clamp easily attaches to headboards.</p>	<p>For patients with no motor skills</p>
<b>PAD/CALL Nurse Call</b>		<p>PadCall is a pneumatic air-activated call cord designed for patients who have difficulty activating standard call cords.</p>	<p>For use by patients with sufficient motor skills to press or lean on the pad. Also great for geriatric patients who cannot operate a regular call cord.</p>
<b>PRESS CALL Nurse Call</b>		<p>PressCall is a pneumatic air-activated call cord designed for patients who have difficulty activating standard call cords. PressCall has an oval-shaped, soft vinyl bulb design that fits easily into the hand.</p>	<p>For use by patients with sufficient motor skills to press to press on the bulb. Also great for geriatric patients who cannot operate a regular call cord.</p>

<b>EZ Call</b>		It can be placed anywhere for activation including cheek, foot, head, etc. Large tented design allows for flexibility and ease of use	This call bell is an excellent option for patient who have limited hand dexterity.
<b>Big Red</b>		<ul style="list-style-type: none"> <li>• Large surface for pushing</li> <li>• Black base makes it easy to push bell against</li> <li>• Sensitive to touch</li> </ul>	<ul style="list-style-type: none"> <li>• Useful for patients with low vision due to large size &amp; bright color</li> <li>• Useful for patients with poor coordination due to large size</li> </ul>
<b>Buddy Button or Jelly Bean (assess as 1 device)</b>		<ul style="list-style-type: none"> <li>• Small size for easy placement around patient</li> <li>• Both sensitive to touch, though blue bell (without black base) is slightly more sensitive to touch</li> </ul>	<ul style="list-style-type: none"> <li>• This call bell is an excellent option for patient who have limited hand dexterity.</li> </ul>
<b>Locline</b>		<ul style="list-style-type: none"> <li>• 2 ft tubing with clamp on one end to clip to a surface &amp; mounting disk on other end</li> <li>• Call bell can be attached to mounting disk with velcro &amp; hook</li> <li>• Linked beads can be removed to shorten arm, but requires force</li> </ul>	<ul style="list-style-type: none"> <li>• Useful to mount call bell on in order to position call bell close to body part the pt can consistently move to activate the bell</li> <li>• Can be used with any of the call bells pictured above to get &amp; maintain them in the best position</li> </ul>

**QUAD BELL  
Adaptor  
For Floors that can  
not connect to the  
bed**



- This adaptor can be used for:
- Breath Activated
  - PAD/CALL Nurse Call
  - PRESS CALL